



ReSound GN

ReSound Forte™

Get a good start  
with the  
ReSound  
Smart 3D app



# How to download the ReSound Smart 3D app

## Download to your Android device

**STEP 1:** Make sure your mobile device is compatible with your hearing instruments. You can find a complete list of compatible devices at [www.future-resound.com](http://www.future-resound.com).

**STEP 2:** Make sure that your Android device is running the recommended Android OS version 5.0 - Lollipop or later versions.

**STEP 3:** Download the ReSound Smart 3D app from Google Play:

1. Tap the Google Play icon.
2. Type “ReSound Smart 3D app” in the search field.
3. Tap “Install” to load the app on your phone.



## Download to your Apple device

**STEP 1:** Make sure your Apple® device is compatible with your Made for iPhone hearing aids.

Visit [www.future-resound.com](http://www.future-resound.com) for a full list of compatible devices and iOS software.

**STEP 2:** Download the ReSound Smart 3D app.

1. Tap the App Store icon.
2. Type “ReSound Smart 3D” in the search field. If you are on an iPad, change the search criteria in the top left-hand corner to iPhone only.
3. Once the ReSound Smart 3D app screen is displayed, tap “GET.” (It will then change to “INSTALL.”)
4. Tap “INSTALL” to load the app on your device.

Note: Although this is a free app, you will be prompted to sign into your iTunes account.

5. Type your Apple ID and password to complete the download process.

You are now ready to pair your hearing aids with your iPhone®, iPad® or iPod touch®. If you need assistance with your device, please contact your local Apple store to make an appointment.

### **You are now ready to pair your hearing instruments**

For instructions on how to pair your hearing instruments with your mobile device, please read the Quick Guide for Connectivity that you received with your hearing instruments. You can also find the Quick Guide at [www.future-resound.com](http://www.future-resound.com). If you need assistance, please contact your hearing care center.

# My information

Please provide the following information and bring this brochure with you to your next appointment.

My smartphone is: \_\_\_\_\_

My Apple ID/Google Play is: \_\_\_\_\_

User name: \_\_\_\_\_

Password: \_\_\_\_\_

My other streaming devices: \_\_\_\_\_

To schedule a hearing evaluation or for more information contact:

## Manufacturer according to FDA:

### ReSound North America

8001 East Bloomington Freeway  
Bloomington, MN 55420  
1.800.882.3636  
future-resound.com

## Manufacturer according to Health Canada:

### ReSound Canada

303 Supertest Road  
Toronto, Ontario M3J 2M4  
1.888.737.6863  
future-resound.com

© 2017 GN ReSound A/S. All rights reserved. ReSound is a trademark of GN ReSound A/S. Apple, the Apple logo, iPhone, iPad, iPod and iPod touch are trademarks of Apple Inc., registered in the U.S. and other countries. App Store is a service mark of Apple Inc.

ReSound GN



MK604915

MK604915 Rev A 2017.09